


Bioline™ DENGUE NS1 Ag

DENGUE NS1 ANTIGEN TEST

Bioline™ Dengue NS1 Ag test is an *in vitro* immunochromatographic, assay designed to detect Dengue virus NS1 antigen in human serum, plasma or whole blood.

- Diagnosis of early acute dengue infection from 1 day onset of fever
- Specimen : Serum, plasma or whole blood (100 µl)
- Test result : 15~20 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 92.4 %, Specificity 98.4 %

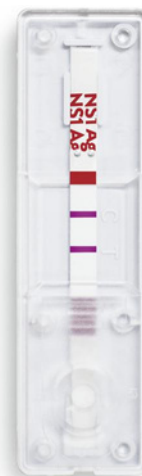


MATERIALS PROVIDED

- Test device
- Disposable dropper


SIMPLE PROCEDURE

1 Add Specimen


Dispense 3 drops (100 µl) of serum, plasma or whole blood into the round specimen well "S".

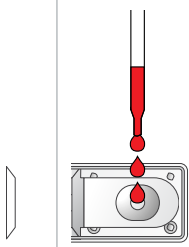

Wait 15-20 mins.

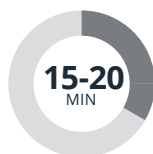

RESULTS INTERPRETATION

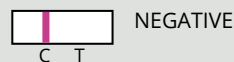


NEGATIVE



POSITIVE



INVALID

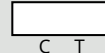

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Dengue NS1 Ag	11FK50	Device	Serum/Plasma/Whole blood	25T/ kit