


Bioline™ DENGUE NS1 Ag

DENGUE NS1 ANTIGEN TEST

Bioline™ Dengue NS1 Ag test is an *in vitro* immunochromatographic, assay designed to detect Dengue virus NS1 antigen in human serum, plasma or whole blood.

- Diagnosis of early acute dengue infection from 1 day onset of fever
- Specimen : Serum, plasma or whole blood (100 µl)
- Test result : 15~20 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 92.4 %, Specificity 98.4 %

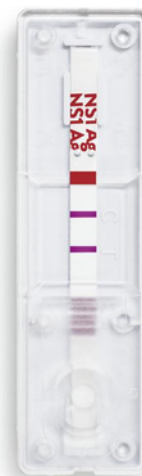


MATERIALS PROVIDED

- Test device
- Disposable dropper


SIMPLE PROCEDURE

1 Add Specimen


Dispense 3 drops (100 µl) of serum, plasma or whole blood into the round specimen well "S".

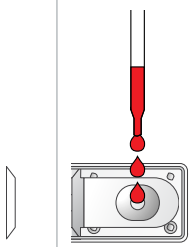

Wait 15-20 mins.

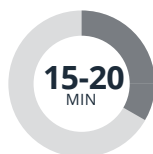


RESULTS INTERPRETATION

 NEGATIVE
C T

 POSITIVE
C T

 INVALID
C T

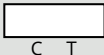

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Dengue NS1 Ag	11FK50	Device	Serum/Plasma/Whole blood	25T/ kit