Information leaflet:

STARBALM® **AQUA GEL HEAT PATCH**

*Read the information leaflet
carefully before use.*

STARBALM® AQUA GEL HEAT PATCH

STARBALM® Aqua Gel Heat Patch is a warm plaster and is a medical device.

STARBALM® Aqua Gel warm plasters can be used for symptomatic relief of muscular aches and painful joints. The STARBALM® Aqua Gel Heat Patches are flexible, with excellent affinity to the skin and comfortable to use on shoulders, back, knees and other moving body parts.

Usage

Make sure the area of application is cleaned and dry. Peel off the protective film and apply plaster to the affected area.

Each plaster can be used once.

Warnings

For external use only and apply on intact skin. Do not use around the eyes, mucous membranes, sensitive areas and/or irritated skin or hematomas. Do not use in case of hypersensitivity to any of the ingredients. Immediately stop using when irritation of the skin or an allergic reaction occurs. If symptoms persist, consult your doctor. The STARBALM® Aqua Gel Heat Patches should not be used on children under 12 years of age and on people with allergic reactions to medication and cosmetics. Do not use, without consulting your doctor, if you are pregnant or breast-feeding.

Storage

Keep out of reach of children. Store patches cool, dark and away from heat and light sources.

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